# JETLAVAGE

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Dongguan Kaiser Technology Co., Ltd.  
No.81, Sanjiang Industrial District, Hengli  
Town, Dongguan, Guangdong, China

MedNet GmbH  
Borkstrasse 10, 48163 Muenster  
Germany

jet lavage

# JETLAVAGE

With adapter  
incl. short and long tip for knee and hip surgery  
Art. No. MST2500Z-ACB/ Blister Packaging

mst  
medical service team

www.mst-gmbh.eu

## Manual

### General Description:

The Pulse Lavage is a fully disposable single use instrument, designed for use in orthopedic surgical procedures, trauma and general wound care.

This pack includes:


- 1 Hand piece
- 1 Small cone tip MSTST01-Z
- 1 Coaxial Canal tip MSTCT02-Z

### Specifications:

- DC 9V over AC Adapter Art. Nr. MST2500N-AC
- Tube length 3 meter
- High speed flow rate up to 1200ml/min
- Low speed flow rate 500ml/min
- Pressure up to 15 PSI

### Instructions for Use:

- Peel open the pack and remove all of the items.
- Attach the required tip to the handle of the gun. Push the security clip (A) up to open and push down to fix the tip.  
Note: When using the coaxial tip (CT 02 Z) ensure the cap is removed before use.
- Circulation person plug in AC adapter power cord to AC outlet source first.
- Attach the irrigation tip (D) to the irrigation source.
- Attach the suction tube end (C) to the suction carrier source.
- Remove the protector from the toggle switch (E) and depress top for high speed fluid delivery. Depress down for low speed fluid delivery. Center the toggle switch for no fluid delivery.
- The suction tube has a clip for reducing suction volume.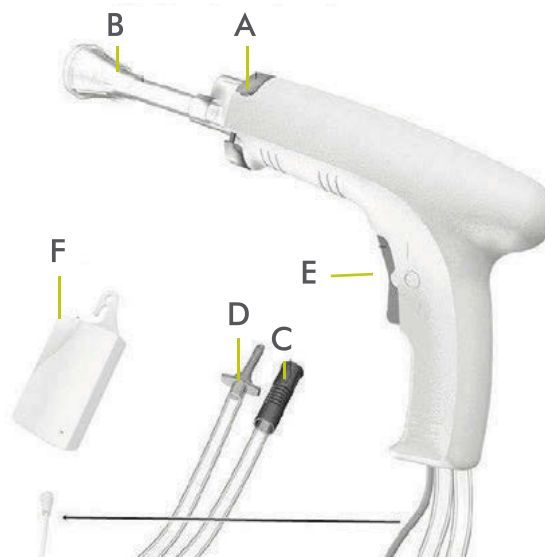


### Caution:

International law restricts this device to sale by or on the order of a physician.

### Warnings:

- Sterile only if package is unopened and undamaged.
- Ensure that the device is disposed of safely and in accordance with all current national and international waste disposal directives.
- Store in a cool dry place.
- Single use only! DO NOT RE-STERILIZE

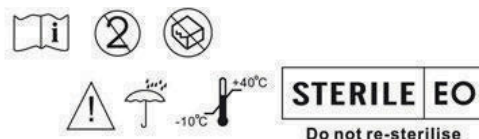**NOTE**  
DO NOT dispose battery in fire  
and NOT rechargeable.


CE  
0197

MSTCT02-Z - Long Tip

MSTST01-Z - Short Tip

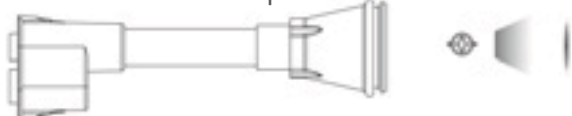Optional: MSTBT04-Z - Femur Canalbrush

Distributed by:  
mst medical-service gmbh  
eichkampredder 9  
24217 schönberg

Telephone: +49 4344 818 36-20  
Telefax: +49 4344 818 36-29  
E-Mail: info@mst-gmbh.eu

Dongguan Kaiser Technology Co., Ltd.  
No.81, Sanjiang Industrial District, Hengli  
Town, Dongguan, Guangdong, China

EC REP

MedNet GmbH  
Borkstrasse 10, 48163 Muenster  
Germany

jet lavage