FEMURBÜRSTE / femoral brush

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GB General Description:

The femoral brush has been designed to remove fibro-cartilaginous and granulation tissue, bone and cement fragments, before scouring the surface to enhance fixation at the bone interface.
The femoral brush is supplied sterile in boxes of 10 pieces.

Specification:

- Stiff polyester brush to scour all surfaces
- Helical action removes debris efficiently
- Plastic sleeve prevents glove tear and protects the surrounding soft tissue
- Single use to minimize the risk of infection
- Choice of handle or power driven

Measurements:

Total length:	280 mm
Length of femur brush:	110 mm
Ø femur brush:	20 mm
Ø hand piece/plastic sleeve:	8 mm
Ø connection to handle/ power tool:	6,5 mm

Instructions for use:

1. Open peel pack and remove the femoral brush.
2. Attach the femoral brush to a handle or to a Jacobs Chuck of your power tool.
3. After usage please dispose the item accordingly.

Warnings:

- This item should only be used by a physician. Package only sterile if unopened and undamaged.
- Single use only. Do not re-sterilized.

CE
0197

Vertrieb durch: / Distributed by:
mst medical-service gmbh
eichkampredder 9
24217 schönberg

Telefon: +49 4344 818 36-20
Telefax: +49 4344 818 36-29
E-Mail: info@mst-gmbh.eu

Dongguan Kaiser Technology Co., Ltd.
No.81, Sanjiang Industrial District, Hengli
Town, Dongguan, Guangdong, China

EC REP

MedNet GmbH
Borkstrasse 10, 48163 Muenster
Germany